

What to do if client has Mini Microphone?



During clinic visit or whilst trouble shooting any connection issues between MM and Nucleus 7 (or Baha 5), check the Mini Microphone the client uses – is the serial number lower than 1481000151?

- No? No need to update the Mini Microphone!
- Yes? Follow the attached instructions and update the Mini Microphone.

I

How to update the firmware on the MM



Get Computer/Software Ready

1. Open GN Resound fitting software - Smart Fit 1.2
2. The application you'll need to run is known as the GN accessory updater and is built into the software

Check Serial Number of the Mini Microphone

If the Mini Microphone is in its original packaging, check the label for the serial number.

If the serial number is lower than 1481000151, the device will need a firmware update.





How to update the firmware on the MM



Check Serial Number of the Mini Microphone

Remove the silver cap to check the serial number of the Mini Microphone.

If the serial number is lower than 1481000151, the device will need a firmware update.



How to update the firmware on the MM



- Click Accessory Updater in the GN software



- Attach the mini-USB cable to the MM and to the computer



How to update the firmware on the MM



- Click 'update'. Time: approximately 30 seconds



- Firmware updated!

